## WordConnect für sonoGDT

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The screenshot shows a software interface with a text window on the left and a grid of images on the right. The text window contains the following information:

```
NachName = Testpatien, Karl  
ID = 1234  
geboren = 30.12.1899  
Patient Characteristics Body Surface Area = 2,22 m2  
Patient Characteristics Patient Height = 1,78 m  
Patient Characteristics Patient Weight = 104,33 kg  
Patient Characteristics Exam date = 30.07.2013  
Patient Characteristics Study date = 30.07.2013  
Patient Characteristics Findings Left Ventricle Measurement Group Left V  
Patient Characteristics Findings Left Ventricle Measurement Group Intern  
Patient Characteristics Findings Left Ventricle Measurement Group Intern  
Patient Characteristics Findings Left Ventricle Measurement Group Left V  
Patient Characteristics Findings Left Ventricle Measurement Group Left V  
Patient Characteristics Findings Left Ventricle Measurement Group Left V
```

The image grid on the right shows a date '30.07.13' at the top, a document icon with a checkmark, and several ultrasound images of a heart in different views.

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The screenshot shows a software interface similar to the previous one, but with a simplified text window. The text window contains the following information:

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The image grid on the right is identical to the previous screenshot, showing a date '30.07.13', a document icon, and several ultrasound images of a heart.

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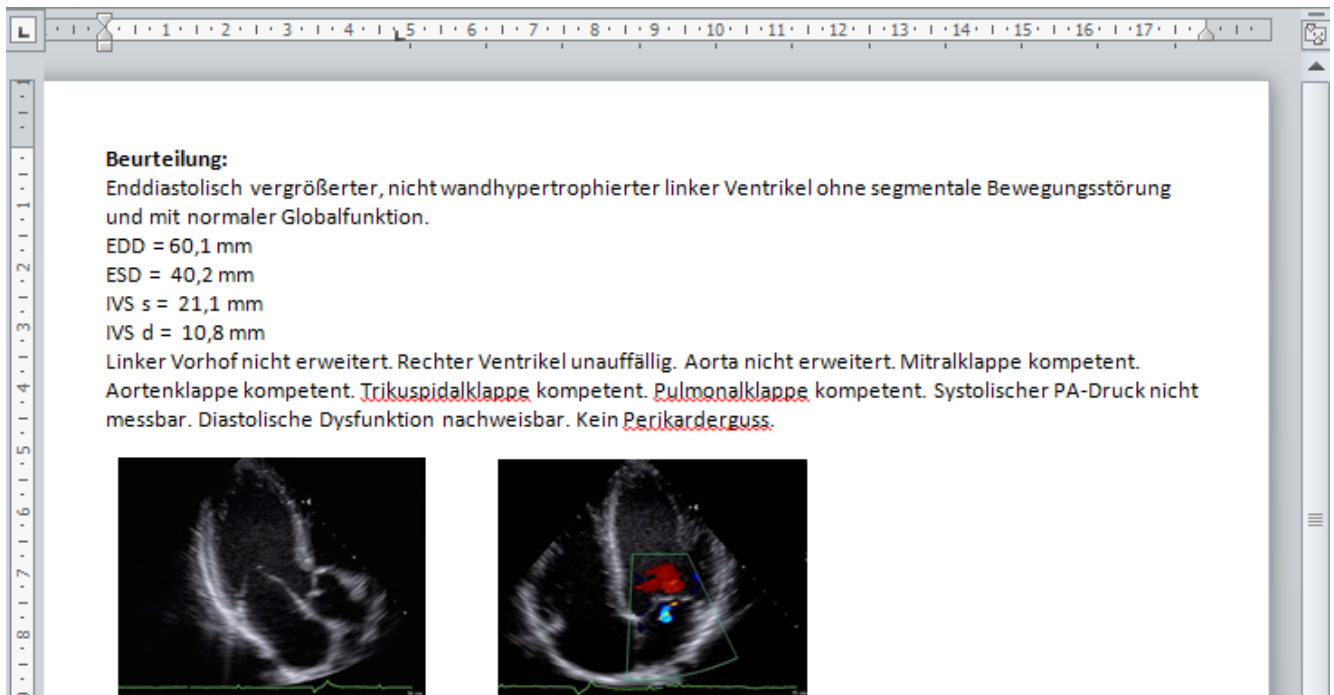
The screenshot shows a Microsoft Word document with a ruler at the top and a vertical scroll bar on the left. The text in the document is as follows:

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Below the text are two side-by-side echocardiography images. The left image is a grayscale B-mode image of the heart. The right image is a color Doppler image of the same heart, showing blood flow in red and blue. A green box is drawn around the heart in the right image.

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